

# E | F35

## STEP-BY-STEP DOCKING

- Always make sure the F35 is switched off before docking and undocking.
- Always engage the wheelchair wheellocks.
- Ensure the wheelchair is fully connected with the F35 before use.
- As with all moving parts, there is a risk that fingers and / or clothing could become trapped.
- Be extra careful when operating the docking clamps.



1 Insert the sub-frame into the docking clamps.



2 Now rotate the tommy screws of the clamps in a clockwise direction until they are hand tight and the intermediate sub-frame is securely held in the clamps.



3 Take the pull device and insert the docking pin ends into the central bracket of the sub-frame.




4 Push the handlebar forwards/ upwards until the locking bolt automatically clicks into place. Only release the wheelchair wheellocks once F35 is locked in place.



Sunrise Medical cares about your safety. Please read your owner's manual for safe operating instructions and maintenance guidelines.

Need more help? Watch the Docking/Undocking video, plus others on charging, drive settings, and more.

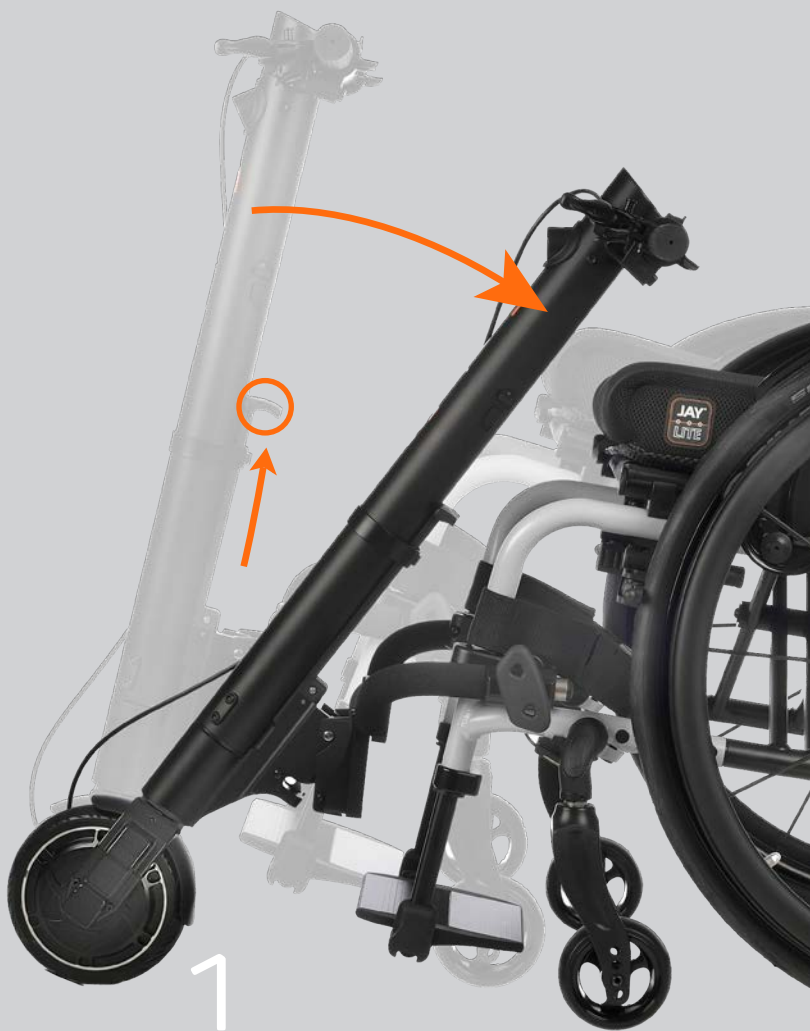

1  
Pull and hold the docking release lever on the main tube. Now pull the handlebar gently backwards/downwards.

# E | F35

## STEP-BY-STEP UNDOCKING

- Always make sure the F35 is switched off before docking and undocking.
- Always engage the wheelchair wheellocks.
- Ensure the wheelchair is fully connected with the F35 before use
- As with all moving parts, there is a risk that fingers and / or clothing could become trapped.
- Be extra careful when operating the docking clamps.



2  
Now lower the pull device slowly until the castors of the wheelchair touch the floor. To remove the F35 from the sub-frame only, steps 1 and 2 can be performed independently of steps 3 and 4.



3  
Undo the tommy screws of the clamps on both sides and pull the sub-frame until the tubes of the intermediate sub-frame have been pulled completely out of the clamps.



4  
Take out the sub-frame from the docking clamps.