



GS



## Zippie<sup>®</sup> GS SE

- with transit option       without transit option

**Supplier:** This manual must be given to the user of this wheelchair.

**User:** Before using this wheelchair read this entire manual and save for future reference.

Owner's Manual

P/N 930377 Rev. G

**SUNRISE MEDICAL LISTENS**

Thank you for choosing a Quickie wheelchair. We want to hear your questions or comments about this manual, the safety and reliability of your chair, and the service you receive from your Sunrise supplier. Please feel free to write or call us at the address and telephone number below:

**SUNRISE MEDICAL (US) LLC**  
 Customer Service Department  
 12002 Volunteer Blvd.  
 Mount Juliet, TN 37122, USA  
 (800) 333-4000



[www.SunriseMedical.com/register](http://www.SunriseMedical.com/register)

Be sure to return your warranty card, and let us know if you change your address. This will allow us to keep you up to date with information about safety, new products and options to increase your use and enjoyment of this wheelchair. If you lose your warranty card, call or write and we will gladly send you a new one.

**FOR ANSWERS TO YOUR QUESTIONS**

Your authorized supplier knows your wheelchair best, and can answer most of your questions about chair safety, use and maintenance.

For future reference, please fill in the following:

**Supplier:** \_\_\_\_\_

**Address:** \_\_\_\_\_

**Telephone:** \_\_\_\_\_

**Serial #:** \_\_\_\_\_ **Date/Purchased:** \_\_\_\_\_

**ADDITIONAL INFORMATION YOU SHOULD KNOW**


No component of this chair was made with Natural Rubber Latex.

**DISPOSAL AND RECYCLING INFORMATION**

When this product reaches the end of its life, please take it to an approved collection or recycling point designated by your local or state government. This wheelchair is manufactured using a variety of materials. Your product should not be disposed of as ordinary household waste. You should dispose of your wheelchair properly, according to local laws and regulations. Most materials that are used in the construction of this product are fully recyclable. The separate collection and recycling of your product at the time of disposal will help conserve natural resources and ensure that it is disposed in a manner that protects the environment.

Ensure you are the legal owner of the product prior to arranging for the product disposal in accordance with the above recommendations.

<b>I. INTRODUCTION</b> .....	<b>2</b>	H. Positioning belts (optional) .....	11
<b>II. TABLE OF CONTENTS</b> .....	<b>3</b>	I. Power drive .....	11
<b>III. YOUR CHAIR AND ITS PARTS</b> .....	<b>4</b>	J. Push handles (optional) .....	11
<b>IV. NOTICE– READ BEFORE USE</b> .....	<b>5</b>	K. Quick-release axles .....	11
A. Choose the right chair & safety options .....	5	L. Rear wheels .....	11
B. Review this manual often .....	5	M. Rear wheel locks .....	12
C. Warnings .....	5	N. Modified seat systems .....	12
<b>V. GENERAL WARNINGS</b> .....	<b>5</b>	O. Upholstery fabric .....	12
A. Weight limit .....	5	<b>IX. SET-UP &amp; ADJUSTMENT</b> .....	<b>13</b>
B. Weight training .....	5	A. Backrest .....	13
C. Getting to know your chair .....	5	B. Swing-in/swing-out hangers and footrests (70°, 80°) .....	13
D. To reduce the risk of an accident .....	5	C. Swing-away hangers (60°, 70°, or 90°) .....	13
E. Safety checklist .....	5	D. Articulating legrest (optional) .....	13
F. Changes & adjustments .....	6	E. Elevating and 90° elevating legrests (optional) .....	14
G. Environmental conditions .....	6	F. Kids & adult angle adjustable footplate .....	14
H. Terrain .....	6	G. Heavy duty lift-off hangers (70°, 80°) .....	14
I. Street use .....	6	H. Hanger extensions (2", 4") .....	14
J. Motor vehicle safety .....	6	I. Aluminum footplate .....	15
K. When you need help .....	6	J. 90° footboard .....	15
<b>VI. WARNINGS: FALLS &amp; TIP-OVERS</b> .....	<b>7</b>	K. Platform flip-up footplate .....	15
A. Center of balance .....	7	L. Flip-up armrests.....	15
B. Dressing or changing clothes .....	7	M. Adjustable locking flip-up armrest (optional) .....	15
C. Wheelies .....	7	N. Length-adjustable locking flip-up armrest (optional) .....	16
D. Obstacles .....	7	O. Height-adjustable armrests (optional) .....	16
E. Reaching or leaning .....	7	P. Wheel locks .....	17
F. Moving backward .....	8	Q. Quick-release axles .....	17
G. Ramps, slopes & side hills .....	8	R. Hub lock .....	17
H. Transfer .....	8	S. Caster forks .....	17
I. Curbs & steps .....	8	T. Caster plates .....	18
J. Stairs .....	9	U. Anti-tip tubes - rear and front .....	18
K. Escalators .....	9	V. Rear wheel adjustments .....	19
<b>VII. WARNINGS: FOR SAFE USE</b> .....	<b>9</b>	W. Growing seat pan .....	20
A. Transit use .....	9	X. Adjustable back upholstery .....	20
B. Learning to do a “wheelie” .....	9	Y. Solid back and seat insert .....	21
C. Descending a curb or single step .....	9	Z. Seat depth .....	21
D. Climbing a curb or single step .....	9	AA. Seat tilt .....	21
E. Climbing stairs .....	10	BB. Cross brace assembly .....	21
F. Descending stairs .....	10	CC. Strut tube assembly .....	21
G. Maintenance .....	10	DD. Folding the chair .....	22
<b>VIII. WARNINGS: COMPONENTS &amp; OPTIONS</b> .....	<b>10</b>	EE. Check-out .....	22
A. Transit option .....	10	<b>X. TROUBLESHOOTING</b> .....	<b>23</b>
B. Anti-tip tubes (optional) .....	10	<b>XI. MAINTENANCE</b> .....	<b>24</b>
C. Armrests .....	10	A. Introduction .....	24
D. Cushions & sling seats .....	10	B. Maintenance chart .....	24
E. Fasteners .....	11	C. Maintenance tips .....	24
F. Footrests .....	11	D. Cleaning .....	24
G. Pneumatic tires .....	11	E. Storage tips .....	24
		<b>XII. SUNRISE LIMITED WARRANTY</b> .....	<b>25</b>

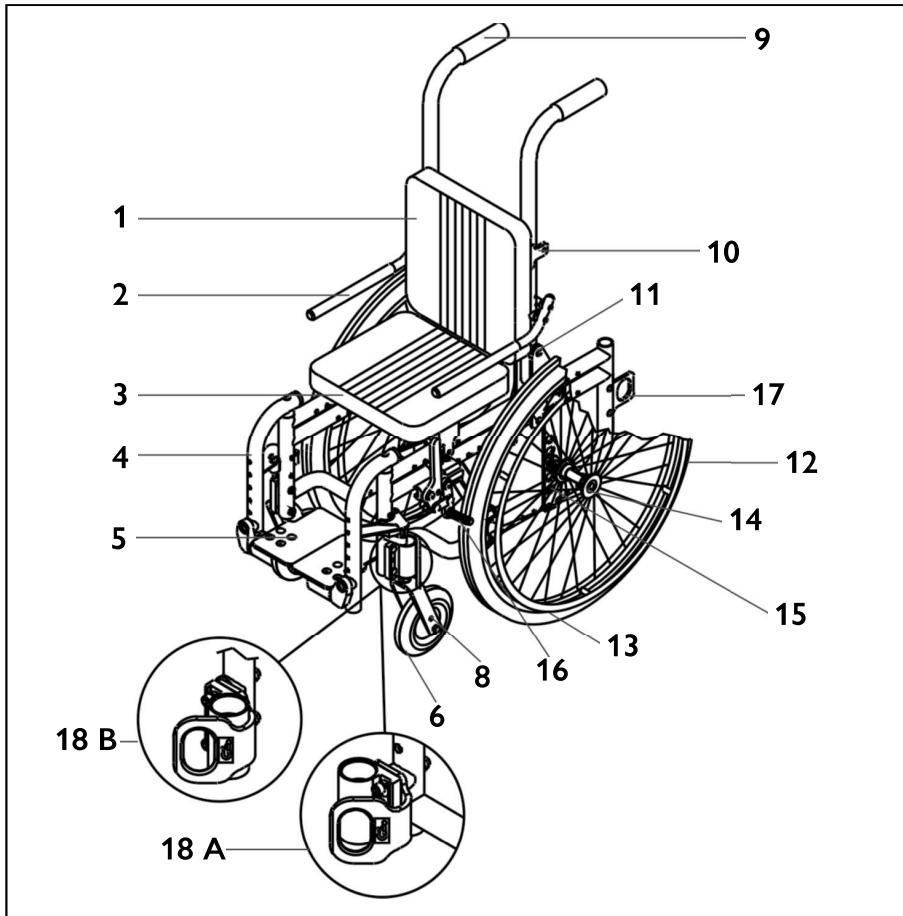

1. Solid back
2. Flip-up armrest
3. Solid seat
4. Swing-away footrest
5. Footplate
6. Caster tire
7. Folding cross-brace (not shown)
8. Caster fork
9. Push handles/folding backrest
10. Solid back hook clamps
11. Backrest pivot plate
12. Rear wheel
13. Handrim
14. Quick-release axle
15. Axle plate
16. Wheel lock
17. Rear securement point (Optional)
- 18 A. Front securement point in forward position (Optional)
- 18 B. Front securement point in trailing position (Optional)

ZIPPIE GS

## A. CHOOSE THE RIGHT CHAIR & SAFETY OPTIONS

Sunrise provides a choice of many wheelchair styles to meet the needs of the wheelchair rider. However, final selection of the type of wheelchair, options and adjustments rests solely with you and your health care advisor. Choosing the best chair and set-up for your safety depends on such things as:

1. Your disability, strength, balance and coordination.
2. The types of hazards you must overcome in daily use (where you live and work, and other places you are likely to use your chair).
3. The need for options for your safety and comfort (such as anti-tip tubes, positioning belts, or special seating systems).

## B. REVIEW THIS MANUAL OFTEN

Before using this chair you, and each person who may assist you, should read this entire manual and make sure to follow all instructions. Review the warnings often, until they are second nature to you.

## C. WARNINGS

The word “**WARNING**” refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The “Warnings” are in four main sections, as follows:

### 1. V – GENERAL WARNINGS

Here you will find a safety checklist and a summary of risks you need to be aware of before you ride this chair.

### 2. VI – WARNINGS – FALLS & TIP-OVERS

Here you will learn how to avoid a fall or tip-over while you perform daily activities in your chair.

### 3. VII – WARNINGS – FOR SAFE USE

Here you will learn about practices for the safe use of your chair

### 4. VIII – WARNINGS – COMPONENTS & OPTIONS

Here you will learn about the components of your chair and options you can select for safety. Consult your authorized supplier and your health care advisor to help you choose the best set-up and options for safe use.

**NOTE** – Where they apply, you will also find “Warnings” in other sections of this manual. Heed all warnings in this section. If you fail to do so a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

## A. WEIGHT LIMIT



### WARNING

**NEVER** exceed the weight limit of 165 pounds (75 kilo-grams), for combined weight of rider and items carried. If you do exceed the limit, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

## B. WEIGHT TRAINING



### WARNING

**NEVER** use this chair for weight training if total weight (rider plus equipment) exceeds 165 pounds. If you do exceed the limit, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

## C. GETTING TO KNOW YOUR CHAIR



### WARNING

Every wheelchair is different. Take the time to learn the feel of this chair before you begin riding. Start slowly, with easy, smooth strokes. If you are used to a different chair, you may use too much force and tip over. If you use too much force, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

## D. TO REDUCE THE RISK OF AN ACCIDENT



### WARNING

1. BEFORE riding, you should be trained in the safe use of this chair by your health care advisor.
2. Practice bending, reaching and transfers until you know the limit of your ability. Have someone help you until you know what can cause a fall or tip-over and how to avoid doing so.
3. Be aware that you must develop your own methods for safe use best suited to your level of function and ability.
4. NEVER try a new maneuver on your own until you are sure you can do it safely.
5. Get to know the areas where you plan to use your chair. Look for hazards and learn how to avoid them.
6. Use anti-tip tubes unless you are a skilled rider of this chair and are sure you are not at risk to tip over.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

## E. SAFETY CHECKLIST



### WARNING

Before Each Use Of Your Chair:

1. Make sure the chair rolls easily and that all parts work smoothly. Check for noise, vibration, or a change in ease of use. (They may indicate low tire pressure, loose fasteners, or damage to your chair).
2. Repair any problem. Your authorized supplier can help you find and correct the problem.
3. Check to see that both quick-release rear axles are locked. When locked, the axle button will “pop out” fully. If not locked, the wheel may come off and cause you to fall.
4. If your chair has anti-tip tubes, lock them in place.

**NOTE** – See “Anti-Tip Tubes” (Section VIII) for times you should not use them. If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

**F. CHANGES & ADJUSTMENTS** **WARNING**

1. If you modify or adjust this chair, it may increase the risk of a tip-over UNLESS you make other changes as well.
2. Consult your authorized supplier BEFORE you modify or adjust your chair.
3. We recommend that you use anti-tip tubes until you adapt to the change, and are sure you are not at risk to tip over.
4. Unauthorized modifications or use of parts not supplied or approved by Sunrise may change the chair structure. This will void the warranty and may cause a safety hazard.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**G. ENVIRONMENTAL CONDITIONS** **WARNING**

1. Use extra care if you must ride your chair on a wet or slick surface. If you are in doubt, ask for help.
2. Contact with water or excess moisture may cause your chair to rust or corrode. This could cause your chair to fail.
  - a. Do not use your chair in a shower, pool or other body of water. The chair tubing and parts are not water-tight and may rust or corrode from the inside.
  - b. Avoid excess moisture (for example, do not leave your chair in a damp bathroom while taking a shower).
  - c. Dry your chair as soon as you can if it gets wet, or if you use water to clean it.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**H. TERRAIN** **WARNING**

1. Your chair is designed for use on firm, even surfaces such as concrete, asphalt and indoor floors and carpeting.
2. Do not operate your chair in sand, loose soil or over rough terrain. This may damage wheels or axles, or loosen fasteners of your chair.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**I. STREET USE** **WARNING**

In most states, wheelchairs are not legal for use on public roads. Be alert to the danger of motor vehicles on roads or in parking lots.

1. At night, or when lighting is poor, use reflective tape on your chair and clothing.
2. Due to your low position, it may be hard for drivers to see you. Make eye contact with drivers before you go forward. When in doubt, yield until you are sure it is safe.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**J. MOTOR VEHICLE SAFETY**

As indicated on the front cover of this user manual, identify whether your chair has been manufactured with the Transit Option installed.

 **WARNING**

**If your chair is not equipped with the Transit Option:**

Zippie wheelchairs do not meet federal standards for motor vehicle seating.

1. NEVER let anyone sit in this chair while in a moving vehicle.
  - a. ALWAYS move the rider to an approved vehicle seat.
  - b. ALWAYS secure the rider with proper motor vehicle restraints.
2. In an accident or sudden stop the rider may be thrown from the chair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.
3. NEVER transport this chair in the front seat of a vehicle. It may shift and interfere with the driver.
4. ALWAYS secure this chair so that it cannot roll or shift.
5. Do not use any chair that has been involved in a motor vehicle accident.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**If your chair is equipped with the Transit Option:**

See section VIII Warnings: Components and Options, Part A: Transit Option.

**K. WHEN YOU NEED HELP** **WARNING**

**For The Rider:**

Make sure that each person who helps you reads and follows all warnings and instructions that apply.

**For Attendants:**

1. Work with the rider's doctor, nurse or therapist to learn safe methods best suited to your abilities and those of the rider.
2. Tell the rider what you plan to do, and explain what you expect the rider to do. This will put the rider at ease and reduce the risk of an accident.
3. Make sure the chair has push handles. They provide secure points for you to hold the rear of the chair to prevent a fall or tip-over. Check to make sure push handle grips will not rotate or slip off.
4. To prevent injury to your back, use good posture and proper body mechanics. When you lift or support the rider or tilt the chair, bend your knees slightly and keep your back as upright and straight as you can.
5. Remind the rider to lean back when you tilt the chair backward.
6. When you descend a curb or single step, slowly lower the chair in one easy movement. Do not let the chair drop the last few inches to the ground. This may damage the chair or injure the rider.
7. To avoid tipping, unlock and rotate anti-tip tubes up, out of the way. ALWAYS lock the rear wheels and lock anti-tip tubes in place if you must leave the rider alone, even for a moment. This will reduce the risk of a tip over or loss of control of the chair.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**A. CENTER OF BALANCE** **WARNING**

The point where this chair will tip forward, back, or to the side depends on its center of balance and stability. How your chair is set up, the options you select and the changes you make may affect the risk of a fall or tip-over.

1. The Most Important Adjustments Are:
  - a. The position of the rear wheels. The more you move the rear wheels forward, the more likely your chair will tip over backward.
2. The Center Of Balance Is Also Affected By:
  - a. A change in the set-up of your chair, including:
    - The distance between the rear wheels.
    - The amount of rear wheel camber.
    - The seat height and seat angle.
    - Backrest angle.
  - b. A change in your body position, posture or weight distribution.
  - c. Riding your chair on a ramp or slope.
  - d. The use of a back pack or other options, and the amount of added weight.
3. To Reduce The Risk Of An Accident:
  - a. Consult your doctor, nurse or therapist to find out what axle and caster position is best for you.
  - b. Consult your authorized supplier BEFORE you modify or adjust this chair. Be aware that you may need to make other changes to correct the center of balance.
  - c. Have someone help you until you know the balance points of your chair and how to avoid a tip-over.
  - d. Use anti-tip tubes.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**B. DRESSING OR CHANGING CLOTHES** **WARNING**

Your weight may shift if you dress or change clothes while seated in this chair.

To reduce the risk of a fall or tip-over:

1. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.
2. Lock anti-tip tubes in place. (If your chair does not have anti-tip tubes, back it up against a wall and lock both rear wheels).

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**C. WHEELIES** **WARNING**

Doing a “wheelie” means: balancing on the rear wheels of your chair, while the front casters are in the air. It is dangerous to do a “wheelie” as a fall or tip-over may occur. However, if you do it safely, a “wheelie” can help you overcome curbs and obstacles.

1. Consult your doctor, nurse or therapist to find out if you are a good candidate to learn to do a “wheelie.”
2. Do not attempt a “wheelie” UNLESS you are a skilled rider of this chair, or you have help.

**NOTE** – See Section VII-B for steps to learn to do a “wheelie.”

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**D. OBSTACLES** **WARNING**

Obstacles and road hazards (such as potholes and broken pavement) can damage your chair and may cause a fall, tip-over or loss of control.

To avoid these risks:

1. Keep a lookout for danger – scan the area well ahead of your chair as you ride.
2. Make sure the floor areas where you live and work are level and free of obstacles.
3. Remove or cover threshold strips between rooms.
4. Install a ramp at entry or exit doors. Make sure there is not a drop off at the bottom of the ramp.
5. To Help Correct Your Center Of Balance:
  - a. Lean your upper body FORWARD slightly as you go UP over an obstacle.
  - b. Press your upper body BACKWARD as you go DOWN from a higher to a lower level.
6. If your chair has anti-tip tubes, lock them in place before you go UP over an obstacle.
7. Keep both of your hands on the handrims as you go over an obstacle.
8. Never push or pull on an object (such as furniture or a door-jamb) to propel your chair.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**E. REACHING OR LEANING** **WARNING**

If you reach or lean it will affect the center of balance of your chair. This may cause you to fall or tip over. When in doubt, ask for help or use a device to extend your reach.

1. NEVER reach or lean if you must shift your weight sideways or rise up off the seat.
2. NEVER reach or lean if you must move forward in your seat to do so. Always keep your buttocks in contact with the backrest.
3. NEVER reach with both hands (you may not be able to catch yourself to prevent a fall if the chair tips).
4. NEVER reach or lean to the rear unless your chair has anti-tip tubes locked in place.
5. DO NOT reach or lean over the top of the seat back. This may damage one or both backrest tubes and cause you to fall.
6. If You Must Reach Or Lean:
  - a. Do not lock the rear wheels. This creates a tip point and makes a fall or tip-over more likely.
  - b. Do not put pressure on the footrests.

**NOTE** – Leaning forward puts pressure on the footrests and may cause the chair to tip if you lean too far.

c. Move your chair as close as you can to the object you wish to reach.

d. Do not try to pick up an object from the floor by reaching down between your knees. You are less likely to tip if you reach to the side of your chair.

e. Rotate the front casters until they are as far forward as possible. This makes the chair more stable.

**NOTE** – To do this: Move your chair past the object you want to reach, then back up alongside it. Backing up will rotate the casters forward.

f. Firmly grasp a rear wheel or an armrest with one hand. This will help to prevent a fall if the chair tips.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

This will help to prevent a fall if the chair tips.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**F. MOVING BACKWARD** **WARNING**

Use extra care when you move your chair backward. Your chair is most stable when you propel yourself forward. You may lose control or tip over if one of the rear wheels hits an object and stops rolling.

1. Propel your chair slowly and smoothly.
2. If your chair has anti-tip tubes, make sure to lock them in place.
3. Stop often and check to be sure your path is clear.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**G. RAMPS, SLOPES & SIDE HILLS** **WARNING**

Riding on a slope, which includes a ramp or side hill, will change the center of balance of your chair. Your chair is less stable when it is at an angle. Anti-tip tubes may not prevent a fall or tip-over.

1. Do not use your chair on a slope steeper than 10%. (A 10% slope means: one foot in elevation for every ten feet of slope length.)
2. Always go as straight up and as straight down as you can. (Do not "cut the corner" on a slope or ramp).
3. Do not turn or change direction on a slope.
4. Always stay in the CENTER of the ramp. Make sure ramp is wide enough that you are not at risk that a wheel may fall over the edge.
5. Do not stop on a steep slope. If you stop, you may lose control of your chair.
6. NEVER use rear wheel locks to try to slow or stop your chair. This is likely to cause your chair to veer out of control.
7. Beware Of:
  - a. Wet or slippery surfaces.
  - b. A change in grade on a slope (or a lip, bump or depression). These may cause a fall or tip-over.
  - c. A drop-off at the bottom of a slope. A drop-off of as small as 3/4 inch can stop a front caster and cause the chair to tip forward.
8. To Reduce The Risk Of A Fall Or Tip-Over:
  - a. Lean or press your body UPHILL. This will help adjust for the change in the center of balance caused by the slope or side hill.
  - b. Keep pressure on the handrims to control your speed on a down slope. If you go too fast you may lose control.
  - c. Ask for help any time you are in doubt.
9. Ramps At Home & Work - For your safety, ramps at home and work must meet all legal requirements for your area. We recommend:
  - a. Width. At least four feet wide.
  - b. Guardrails. To reduce the risk of a fall, sides of ramp must have guardrails (or raised borders at least three inches high).
  - c. Slope. Not more than a 10% grade.
  - d. Surface. Flat and even, with a thin carpet or other non-skid material. (Make sure there is no lip, bump or depression.)
  - e. Bracing. Ramp must be STURDY. You may need bracing so ramp does not "bow" when you ride on it.
  - f. Avoid A Drop Off. You may need a section at the top or bottom to smooth out the transition.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**H. TRANSFER** **WARNING**

It is dangerous to transfer on your own. It requires good balance and agility. Be aware that there is a point during every transfer when the wheelchair seat is not below you. To Avoid A Fall:

1. Work with your health care advisor to learn safe methods.
  - a. Learn how to position your body and how to support yourself during a transfer.
  - b. Have someone help you until you know how to do a safe transfer on your own.
2. Lock the rear wheels before you transfer. This keeps the rear wheels from rolling.
 

**NOTE** – This will NOT keep your chair from sliding away from you or tipping.
3. Make sure to keep pneumatic tires properly inflated. Low tire pressure may allow the rear wheel locks to slip.
4. Move your chair as close as you can to the seat you are transferring to. If possible, use a transfer board.
5. Rotate the front casters until they are as far forward as possible.
6. If you can, remove or swing footrests out of the way.
  - a. Make sure your feet do not catch in the space between the footrests.
  - b. Avoid putting weight on the footrests as this may cause the chair to tip.
7. Make sure armrests are out of the way and do not interfere.
8. Transfer as far back onto the seat surface as you can. This will reduce the risk that the chair will tip or move away from you.
9. Make sure armrests are locked in place prior to transfer to prevent rotation of armrests and loss of control.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**I. CURBS & STEPS** **WARNING**

1. Each person who helps you should read and follow the warnings "For Safe Use" (Section VII).
2. Do not try to climb or descend a curb or step alone UNLESS you are a skilled rider of this chair and:
  - a. You can safely do a "wheelie", and
  - b. You are sure you have the strength and balance to do so.
3. Unlock and rotate anti-tip tubes up, out of the way, so they do not interfere.
4. Do not try to climb a high curb or step (more than 4 inches high) UNLESS you have help. Doing so may cause your chair to exceed its balance point and tip over.
5. Go straight up and straight down a curb or step. If you climb or descend at an angle, a fall or tip-over is likely.
6. Be aware that the impact of dropping down from a curb or step can damage your chair or loosen fasteners.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*



**J. STAIRS****⚠ WARNING**

1. NEVER use this chair on stairs UNLESS you have someone to help you. Doing so is likely to cause a fall or tip-over.
2. Persons who help you should read and follow the warnings “For Attendants”, and the “For Safe Use” (Section VII).

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**K. ESCALATORS****⚠ WARNING**

**NEVER use this chair on an escalator, even with an attendant. If you do, a fall or tip-over is likely.**

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*


## VII. WARNINGS: FOR SAFE USE

**⚠ WARNING**

**Before you assist a rider, be sure to read the warnings “For Attendants” (p. 12) and follow all instructions that apply. Be aware that you will need to learn safe methods best suited to your abilities.**

**A. TRANSIT USE****⚠ WARNING**

1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraint.
2. If the wheelchair is used for transport, it must be used in a forward-facing seating position.
3. The rider must not weigh more than 165 lbs.
4. Figure 1 shows the location of the wheelchair securement points.

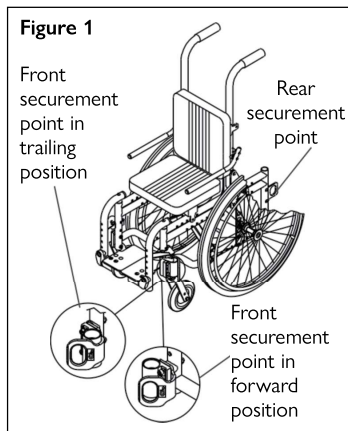

5. Use only with Wheelchair Tiedown and Occupant Restraint Systems (WTORS) that have been installed in accordance with the manufacturer’s instructions and SAE J2249.
6. Attach WTORS to securement points in accordance with the manufacturer’s instructions and SAE J2249.
7. Attach occupant restraints in accordance with the manufacturer’s instructions and SAE J2249.

**NOTE** – To obtain a copy of SAE J2249 Wheelchair Tiedown and Occupant Restraint Systems for Use in Motor Vehicles, please contact: SAE International - 400 Commonwealth Drive - Warrendale, PA 15096 -412.776.4970.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**B. LEARNING TO DO A “WHEELIE”****⚠ WARNING**

Follow these steps to help the rider learn to do a “wheelie”:

1. Read and follow the warnings for “WHEELIES”. (See Section VI-C),
2. Stand at the rear of the chair. You must be able to move with the chair to prevent a tip-over.
3. Keep your hands BENEATH the push handles, ready to catch the rider if the chair tips back.
4. When you are ready, have the rider place his or her hands on the forward part of the handrims.

5. Have the rider make a quick BACKWARD movement of the rear wheels, quickly followed by a hard FORWARD thrust. (This will tilt the chair up on the balance point of the rear wheels).
6. Have the rider make small movements of the handrims to stay in balance.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**C. DESCENDING A CURB OR SINGLE STEP****⚠ WARNING**

Follow these steps to help a rider descend a curb or single step going BACKWARD:

1. Stay at the rear of the chair.
2. Several feet before you reach the edge of the curb or step, turn the chair around and pull it backward.
3. While looking over your shoulder, carefully step back until you are off the curb or stair and standing on the lower level.
4. Pull the chair toward you until the rear wheels reach the edge of the curb or step. Then allow the rear wheels to slowly roll down onto the lower level.
5. When the rear wheels are safely on the lower level, tilt the chair back to its balance point. This will lift the front casters off the curb or step.
6. Keep the chair in balance and take small steps backward. Turn the chair around and gently lower front casters to the ground.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**D. CLIMBING A CURB OR SINGLE STEP****⚠ WARNING**

Follow these steps to help the rider climb a curb or single step going FORWARD:

1. Stay behind the chair.
2. Face the curb and tilt the chair up on the rear wheels so that the front casters clear the curb or step.
3. Move forward, placing the front casters on the upper level as soon as you are sure they are past the edge.
4. Continue forward until the rear wheels contact the face of the curb or step. Lift and roll the rear wheels to the upper level.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**E. CLIMBING STAIRS** **WARNING**

1. Use at least two attendants to move a chair and rider up stairs.
2. Move the chair and rider **BACKWARD** up the stairs.
3. The person at the rear is in control. He or she tilts the chair back to its balance point.
4. A second attendant at the front firmly grasps a non-detachable part of the front frame and lifts the chair up and over one stair at a time.
5. The attendants move to the next stair up. Repeat for each stair, until you reach the landing.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**F. DESCENDING STAIRS** **WARNING**

1. Use at least two attendants to move a chair and rider down stairs.
2. Move the chair and rider **FORWARD** down the stairs.
3. The person at the rear is in control. He or she tilts the chair to the balance point of the rear wheels and rolls it to the edge of the top step.

4. A second attendant stands on the third step from the top and grasps the chair frame. He or she lowers the chair one step at a time by letting the rear wheels roll over the stair edge.
5. The attendants move to the next stair down. Repeat for each stair, until you reach the landing.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**G. MAINTENANCE** **WARNING**

1. Inspect and maintain this chair strictly per maintenance chart.
2. If you detect a problem, make sure to service or repair the chair before use.
3. At least once a year, have a complete inspection, safety check and service of your chair made by an authorized supplier.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others. For additional maintenance information see Section XI: Maintenance.*

## VIII. WARNINGS: COMPONENTS &amp; OPTIONS

**A. TRANSIT OPTION** **WARNING**

1. Use only Wheelchair Tiedown and Occupant Restraint Systems (WTORS) which meet the requirements of SAE J2249 Recommended Practice - Wheelchair Tiedown and Occupant Restraint Systems For Use in Motor Vehicles. Do not use WTORS designed to rely on the wheelchair structure to transfer occupant restraint loads to the vehicle.
2. The wheelchair has been dynamically tested in a forward-facing mode for a 48-km/h frontal impact test. The wheelchair must be forward-facing during transport.
3. In order to reduce the potential of injury to vehicle occupants, wheelchair-mounted accessories, such as trays and respiratory equipment, should be removed and secured separately.
4. Postural supports and positioning devices should not be relied on for occupant restraint.
5. Do not alter or substitute wheelchair frame parts, components of seating.
6. Sudden stops or impacts can structurally damage your chair. Chairs involved in such incidents should be replaced.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**B. ANTI-TIP TUBES (OPTIONAL)** **WARNING**

Anti-tip tubes can help keep your chair from tipping over backward in most normal conditions.

1. Sunrise Recommends Use Of Anti-Tip Tubes:
  - a. **UNLESS** you are a skilled rider of this chair and are sure you are not at risk to tip over.
  - b. Each time you modify or adjust your chair. The change may make it easier to tip backward. Use anti-tip tubes until you adapt to the change, and are sure you are not at risk to tip over.

2. When locked in place (in the “down” position) anti-tip tubes should be **BETWEEN** 1 1/2 to 2 inches off the ground.
  - a. If set too **HIGH**, they may not prevent a tip-over.
  - b. If set too **LOW**, they may “hang up” on obstacles you can expect in normal use. If this occurs, you may fall or your chair may tip over.
3. Keep Anti-Tip Tubes Locked In Place **UNLESS**:
  - a. You have an attendant; or
  - b. You have to climb or descend a curb, or overcome an obstacle, and can safely do so without them. At these times, make sure anti-tip tubes are up, out of the way.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**C. ARMRESTS** **WARNING**

**Armrests detach and will not bear the weight of this chair.**

1. **NEVER** lift this chair by its armrests. They may come loose or break.
2. Lift this chair only by non-detachable parts of the main frame.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**D. CUSHIONS & SLING SEATS** **WARNING**

1. Quickie sling seats and standard foam cushions are not designed for the relief of pressure.
2. If you suffer from pressure sores, or if you are at risk that they will occur, you may need a special seat system or a device to control your posture. Consult your doctor, nurse or therapist to find out if you need such a device for your well-being.
3. When using the growing seat pan (optional), when removing and reattaching the seat pan, check for engagement by pulling up lightly on the seat pan. It should not move upward.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**E. FASTENERS** **WARNING**

Many of the screws, bolts and nuts on this chair are special high-strength fasteners. Use of improper fasteners may cause your chair to fail.

1. ONLY use fasteners provided by an authorized supplier (or ones of the same type and strength, as indicated by the markings on the heads).
2. Over- or under-tightened fasteners may fail or cause damage to chair parts.
3. If bolts or screws become loose, tighten them as soon as you can.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**F. FOOTRESTS** **WARNING**

1. At the lowest point, footrests should be AT LEAST 2 1/2 INCHES off the ground. If set too LOW, they may “hang up” on obstacles you can expect to find in normal use. This may cause the chair to stop suddenly and tip forward.
2. To Avoid A Trip Or Fall When You Transfer:
  - a. Make sure your feet do not “hang up” or get caught in the space between the footrests.
  - b. Avoid putting weight on the footrests, as the chair may tip forward.
3. NEVER lift this chair by the footrests. Footrests detach and will not bear the weight of this chair. Lift this chair only by non-detachable parts of the main frame.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**G. PNEUMATIC TIRES** **WARNING**

Proper inflation extends the life of your tires and makes your chair easier to use.

1. Do not use this chair if any of the tires is under- or overinflated. Check weekly for proper inflation level, as listed on the tire sidewall.
2. Low pressure in a rear tire may cause the wheel lock on that side to slip and allow the wheel to turn when you do not expect it.
3. Low pressure in any of the tires may cause the chair to veer to one side and result in a loss of control.
4. Over-inflated tires may burst.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**H. POSITIONING BELTS (OPTIONAL)** **WARNING**

Use positioning belts ONLY to help support the rider’s posture. Improper use of these belts may cause severe injury to or death of the rider.

1. Make sure the rider does not slide down in the wheelchair seat. If this occurs, the rider may suffer chest compression or suffocate due to pressure from the belts.
2. The belts must be snug, but must not be so tight that they interfere with breathing. You should be able to slide your open hand, flat, between the belt and the rider.
3. A pelvic wedge or a similar device can help keep the rider from sliding down in the seat. Consult with the rider’s doctor, nurse or therapist to find out if the rider needs such a device.
4. Use positioning belts only with a rider who can cooperate. Make sure the rider can easily remove the belts in an emergency.

5. NEVER Use Positioning Belts:
  - a. As a patient restraint. A restraint requires a doctor’s order.
  - b. On a rider who is comatose or agitated.
  - c. As a motor vehicle restraint. In an accident or sudden stop the rider may be thrown from the chair. Wheelchair seat belts will not prevent this, and further injury may result from the belts or straps.

*If you fail to heed these warnings damage to your chair, a fall, tip-over, loss of control may occur or cause severe injury to the rider or others.*

**I. POWER DRIVE** **WARNING**

**Do not install a power drive on any Zippie wheelchair. If you do:**

1. It will affect the center of balance of your chair and may cause a fall or tip-over.
2. It will alter the frame and void the warranty.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**J. PUSH HANDLES (OPTIONAL)** **WARNING**

When you have an attendant, make sure that this chair has push handles.

1. Push handles provide secure points for an attendant to hold the rear of this chair, to prevent a fall or tip-over.
2. Check to make sure push handle grips will not rotate or slip off.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**K. QUICK-RELEASE AXLES** **WARNING**

1. Do not use this chair UNLESS you are sure that both quick-release rear axles are locked. An unlocked axle may come off during use and cause a fall.
2. An axle is not locked until the quick-release button pops out fully. An unlocked axle may come off during use, resulting in a fall, tip-over or loss of control and cause severe injury to the rider or others.
3. Routinely inspect. Replace if worn or bent.
  - Quick-Release Axles should be inspected for function and any signs of wear or bending. Replace as necessary.

 **WARNING**

**Routinely Inspect. Replace if worn or bent.**

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**L. REAR WHEELS** **WARNING**

**A change in set-up of the rear wheels will affect the center of balance of your chair.**

1. The farther you move the rear axles FORWARD, the more likely it is that your chair will tip over backward.
2. Consult your doctor, nurse or therapist to find the best rear axle set-up for your chair. Do not change the set-up UNLESS you are sure you are not at risk to tip over.
3. Adjust the rear wheel locks after you make any change to the rear axles.
  - a. If you fail to do so, the locks may not work.
  - b. Make sure lock arms embed in tires at least 1/8 inch when locked.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

## M. REAR WHEEL LOCKS



### WARNING

Rear wheel locks are **NOT** designed to slow or stop a moving wheelchair. Use the only to keep the rear wheels from rolling when your chair is at a complete stop.

1. NEVER use rear wheel locks to try to slow or stop your chair when it is moving. Doing so may cause you to veer out of control.
2. To keep the rear wheels from rolling, always set both rear wheel locks when you transfer to or from your chair.
3. Low pressure in a rear tire may cause the wheel lock on that side to slip and may allow the wheel to turn when you do not expect it.
4. Make sure lock arms embed in tires at least 1/8 inch when locked. If you fail to do so, the locks may not work.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

## N. MODIFIED SEAT SYSTEMS



### WARNING

Use of a seat system not approved by Sunrise may alter the center of balance of this chair. This may cause the chair to tip over.

1. Do not change the seat system of your chair UNLESS you consult your authorized supplier first.
2. Use of a seat system not approved by Sunrise may affect the folding mechanism of this chair.
3. Use of a seating system not provided by Sunrise is prohibited for transit use.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

## O. UPHOLSTERY FABRIC



### WARNING

1. Replace worn or torn fabric of seat and seat back as soon as you can. If you fail to do so, the seat may fail.
2. Sling fabric will weaken with age and use. Look for fraying or thin spots, or stretching of fabrics at rivet holes.
3. "Dropping down" into your chair will weaken fabric and result in the need to inspect and replace the seat more often.
4. Be aware that laundering or excess moisture will reduce flame retardation of the fabric.

*If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.*

**NOTE** – Setting the wheelchair on a flat surface helps make these procedures easier.


**NOTE ON TORQUE SETTINGS:** A torque setting is the optimum tightening which should be made on a particular fastener. A torque wrench which measures in.-lbs. should be used when tightening fasteners on Sunrise wheelchairs. Unless otherwise noted, use a torque setting of 60 in.-lbs. when setting up this wheelchair.

**A. BACKREST 1**

**1. To Secure The Backrest For Operation:**

- a. Lift backrest to upright position.
- b. When secured, backrest pin will lock into place.

**NOTE** – Zippie series wheelchairs are shipped with the backrest in a folded position.

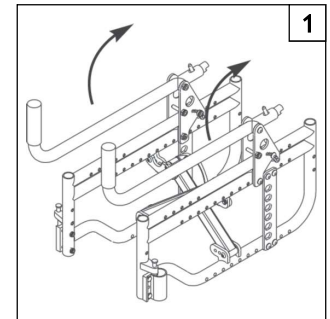

**B. SWING-IN/SWING-OUT HANGERS AND FOOTRESTS (70°, 80°)**

**1. Installation 2**

- a. Place swing-in/swing-out pivot saddle into the receiver on front frame tube with the footrest facing outward from the frame. (2-A)
- b. Rotate the footrest inward until it locks into place on locking bolt. (2-B)



**2. Removal**

- a. To remove footrest, push release latch toward the frame.
- b. Rotate footrest inward and lift. Or, pull footrest away from frame and rotate outward.

**3. Height Adjustment 3**

**NOTE** – It is recommended that you maintain a minimum of 2.0 inches between the lowest point on the footrest and the floor.


- a. Loosen set screw (C) using a 3/16 inch hex key.
- b. Slide footrest extension up or down inside frame tube to desired height.
- c. Tighten set screw. Use a torque setting of 144 in.-lbs (16.27 Nm).

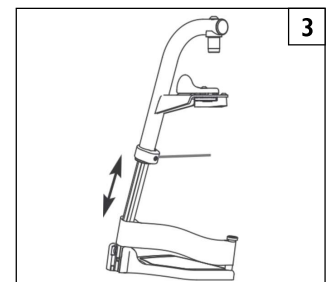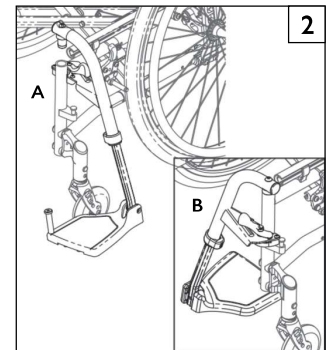

**C. SWING-AWAY HANGERS (60°, 70°, or 90°) 4**

**1. Attaching/Removal**

- a. Place swing-away pivot saddle (D) into the receiver on front frame tube with the footrest facing outward from the frame.
- b. Rotate the footrest inward until it locks into place on locking bolt.
- c. To remove hanger, push release latch (E) toward the frame, rotate footrest outward, and lift.

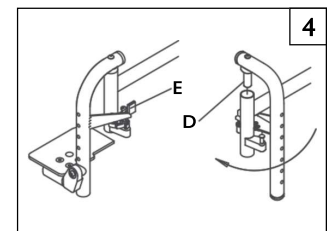


**D. ARTICULATING LEGREST (Optional) 5**

**1. Attaching/Removal**

- a. Place swing-away pivot saddle (F) into the receiver on front frame tube with the footrest facing outward from the frame.
- b. Rotate the footrest inward until it locks into place on locking bolt.
- c. To remove hanger, push release latch (G) toward the frame, rotate footrest outward, and lift.

**2. Elevating**

- a. To raise legrest, lift to desired position. Legrest will automatically lock in place.
- b. To lower legrest, while seated in chair hold lever (H) down, lower legrest to desired position, then release lever. Legrest will automatically lock in place.

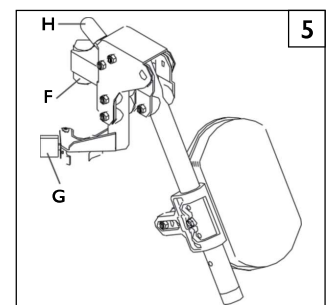



**E. ELEVATING AND 90° ELEVATING LEGRESTS (Optional)** 6 7

**1. Attaching/Removal**

- a. Place swing-away pivot saddle (A) into the receiver on front frame tube with the footrest facing outward from the frame.
- b. Rotate the footrest inward until it locks into place on locking bolt.
- c. To remove hanger, push release latch (B) toward the frame, rotate footrest outward, and lift.

**2. Elevating**

- a. To raise legrest, lift to desired position. Legrest will automatically lock in place.
- b. To lower legrest, while seated in chair hold lever (C) down, lower legrest to desired position, then release lever. Legrest will automatically lock in place.

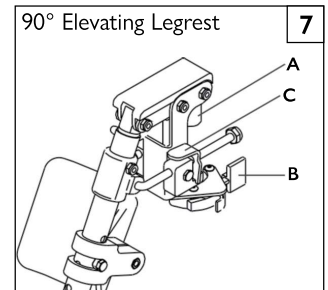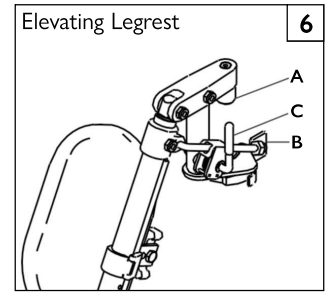

**F. KIDS & ADULT ANGLE ADJUSTABLE FOOTPLATE** 8 9

**1. Height Adjustment**



- a. Remove the retaining bolt from the frame tube (D).
- b. Move footplate assembly to desired height.
- c. Left and right footrest will normally be at equal height.
- d. Replace and tighten bolt.

**2. Angle Adjustment**

- a. Loosen fasteners (E) until the adjustable clamp and footplate rotate freely on the mounting tube.
- b. Adjust the angle of the footplate.
- c. Tighten nuts.

**3. Position Adjustment**


- a. Remove footplate fasteners (E).
- b. Move footplate to the desired location.
- c. Replace and tighten fasteners.

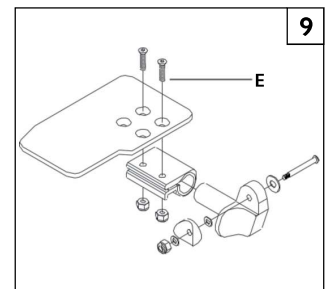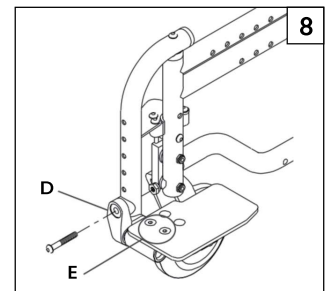

**G. HEAVY DUTY LIFT-OFF HANGERS (70°, 80°)** 10

**1. Attaching/Removal**


- a. Place pivot saddle (F) into the receiver on front frame until locked in place.
- b. To remove hanger, push release mechanism (G) and lift hanger off.

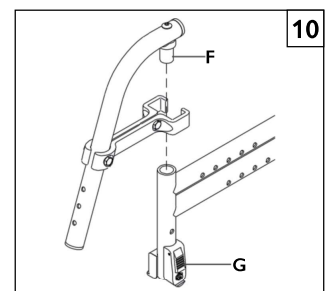

**H. HANGER EXTENSIONS (2", 4")** 11

**1. Installing/Removal**


- a. Remove footplate by removing bolt (H).
- b. Remove end cap (I).
- c. Insert extension (J) into hanger and insert fastener (K).
- d. Reattach footplate to extension by replacing bolt.
- e. To remove extension, reverse this process.

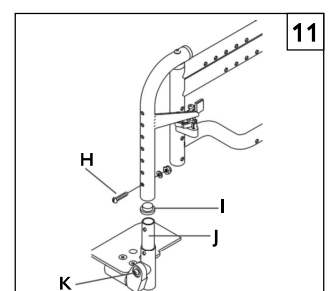

**I. ALUMINUM FOOTPLATE 12**

**1. Height Adjustment**

- Remove set screws (F) from frame tube.
- Slide footplate to desired height.
- The adjustment on each side of the footrest will normally be of equal height.
- Replace and tighten set screws.

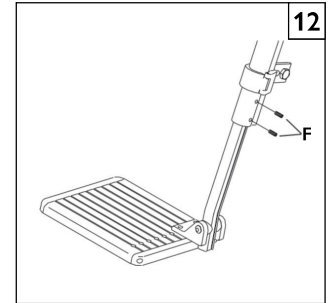

**J. 90° FOOTBOARD 13**

**1. Height Adjustment**


- Loosen the nuts on the footplate clamp assembly (G).
- Slide footplate clamps to desired height.
- Tighten nuts.

**2. Angle Adjustment**

- Loosen nuts under the footplate assembly (H) until the adjustable clamp rotates freely on the supporting tube.
- Adjust the angle of the footplate.
- Slide it forward or back for correct placement.
- Tighten nuts.

**3. Removing Footboard**

- Unscrew the lock knobs (I) and lift out of clamp assembly.

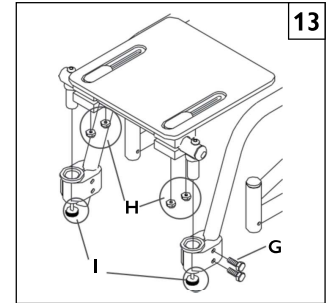


**K. PLATFORM FLIP-UP FOOTPLATE 14**

**1. Height Adjustment**

See Section B-1.

**2. Angle Adjustment**

- Loosen the nuts under the footplate clamp (B) until the adjustable clamp and footplate rotate freely on the mounting tube.
- Adjust the angle of the footplate.
- Tighten nuts.

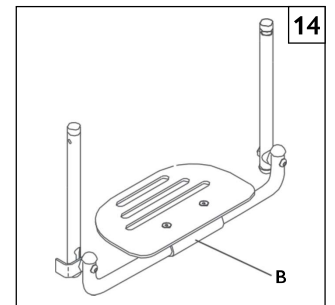


**L. FLIP-UP ARMRESTS 15**

**1. Adjustment**

Flip-up armrests are height adjustable (3") by moving the armrest assemblies up or down in the pre-drilled holes on the backrest frame tubes.

- To adjust armrest upward and downward, loosen cam adjustment bolt (C).
- Rotate armrest to desired position.
- Replace and tighten bolt.

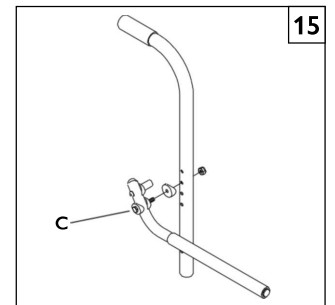


**M. ADJUSTABLE LOCKING FLIP-UP ARMREST (Optional) 16**

**1. Height Adjustment**

- Remove the hex nuts from the armrest (D) and angle adjustment plate (E) to release the armrest and plate.
- Release the bolts from the arm mount (F).
- Move mount up or down the back post in 1-inch increments for desired height, or within the arm mount holes for 1/4" inch increments.
- Replace and tighten nuts and bolts.

**2. Angle Adjustment**

- Loosen the hex nut from the angle adjustment plate (E).
- Tilt the armrest and plate to the desired angle.
- Tighten nut.

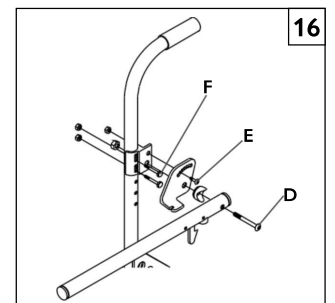

**N. LENGTH-ADJUSTABLE LOCKING FLIP-UP ARMREST (Optional)** 17 18

**1. Height Adjustment**

- a. Remove the hex nuts from the armrest (A) and angle adjustment plate (B) to release the armrest and plate.
- b. Release the bolts from the arm mount (C).
- c. Move mount up or down the back post in 1-inch increments for desired height, or within the arm mount holes for 1/4-inch increments.
- d. Replace and tighten nuts and bolts.



**2. Angle Adjustment**

- a. Loosen the hex nut from the angle adjustment plate.
- b. Tilt the armrest and plate to the desired angle.
- c. Tighten nut.

**3. Length Adjustment (D)** 18

- a. Loosen set screw on underside of adjustable armrest.
- b. Set armrest to desired length.
- c. Tighten set screw.

**NOTE** – Do not set length of armrest longer than frame depth.

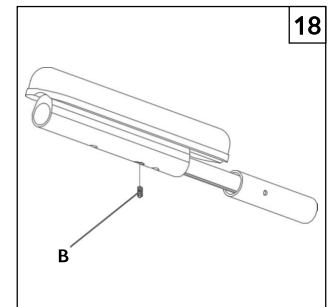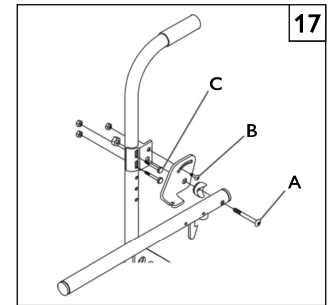

**O. HEIGHT-ADJUSTABLE ARMRESTS (Optional)** 19 19a

**1. Installation**

- a. Slide the outer armpost into the receiver mounted to the wheelchair frame.
- b. The armrest will automatically lock into place.

**2. Height Adjustment**

- a. Rotate release lever to second stop.
- b. Slide armrest pad up or down to desired height.
- c. Return lever to locked position against armpost.
- d. Push arm pad until upper armpost locks firmly into place.

**3. Removing Armrest**

- a. Rotate release lever to first stop and remove the armrest.

**4. Replacing Armrest**

- a. Slide armrest back into receiver.
- b. Return release lever to locked position against armpost.


**5. Adjusting Armrest Receiver Fit**

To tighten or loosen the fit of the outer armpost in the receiver:

- a. Loosen the bolts on the sides of the receiver.
- b. With the armrest in the receiver, squeeze the receiver to achieve the desired fit.
- c. Tighten the four bolts.

**6. Adjusting Inner Armpost Fit**

- a. Two set screws are installed in the outer armpost.
- b. Turn the set screws in or out until the desired fit is achieved.

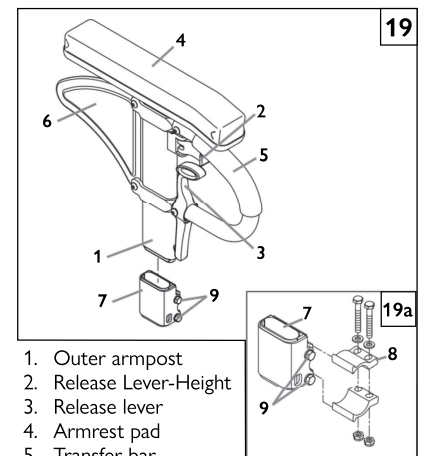

1. Outer armpost
2. Release Lever-Height
3. Release lever
4. Armrest pad
5. Transfer bar
6. Side panel
7. Receiver
8. Clamp
9. Receiver adjustment hardware



**P. WHEEL LOCKS** 20




**WARNING**

Do not operate your wheelchair without wheel locks.

**1. High-Mount Wheel Lock**

- a. Loosen the screws on the top of each clamp (A). Do not attempt to remove one screw at a time.
- b. Slide assembly toward rear wheel until clamp (B) embeds into tire to prevent wheel movement when in locked position.
- c. Adjust angle position.
- d. Tighten screws.

**NOTE** – Use a torque setting of 100 in.-lbs. when setting up wheel locks.

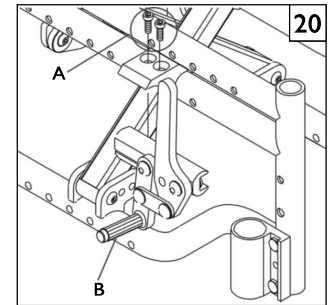

**Q. QUICK-RELEASE AXLES** 21

**1. Attaching/Removal**


- a. Depress the quick-release button on the axle (C).
- b. Slide through wheel hub and into the axle sleeve until it locks.

**NOTE** – The axle is not secured until the outside quick-release button pops out to its fully extended position. The ball bearings on the opposite end of the axle must be visible beyond the end of the axle sleeve.

- c. To remove the wheel hub, depress the quick-release button and lift off the hub.

**2. Axle Length Adjustment**

- a. To adjust the axle you will need a 3/4" wrench to turn the outside axle nuts.
- b. You will also need a 1/2" wrench to lodge the ball bearings on the opposite end of the axle and to prevent the axle from turning.
- c. Turn the outside axle nut counter-clockwise to tighten.
- d. There should only be zero to ten thousandths of an inch (.010") of play between ball bearing and the inside of the axle sleeve.

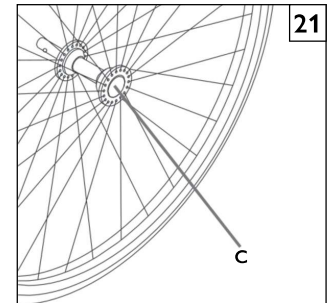

**R. HUB LOCK** 22 23

**1. To operate Hub Lock** 22

- a. To Lock rear wheels, press button (D). Lever (E) will flip outward.
- b. To unlock wheels, press Lever (E) back down until Button (D) pops back out. Each lever operates one side of the chair. To fully lock chair, both buttons must be pushed.



**NOTE** – Do not engage hub lock when chair is in motion.

**2. To adjust the cables** 23

- a. Loosen jam nut (F) using a 10mm wrench.
- b. Turn adjustment screw (G) in or out as necessary. Turning screw upward will loosen cable, turning downward will tighten cable.
- c. Re-tighten jam nut (F) against lever mount.
- d. Repeat steps 1-3 for opposite side of chair as necessary.

**3. To adjust lever position** 23

- a. Remove socket screw (H) and washer (I) from lever and backrest.
- b. Choose desired hole in backrest to mount lever and re-install screw and washer.
- c. Repeat steps 1-2 for opposite side of chair.

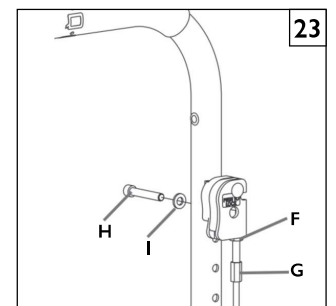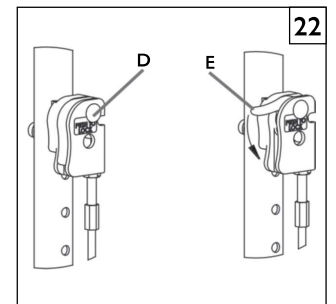


**S. CASTER FORKS** 24

**1. Caster Fork Adjustment**

- a. Loosen the bolt (A) and remove the caster.
- b. Either change caster size and/or position the caster upward or downward in pre-drilled holes (1" apart) on caster fork.
- c. Replace bolt and tighten securely.
- d. Repeat with other caster.


**NOTE** – Two-hole caster forks allow interchanging 5", 6" and 8" casters and adjusting seat height. These changes may affect the floor to seat height.

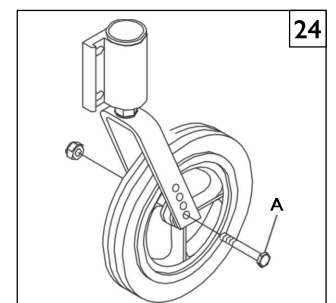

**T. CASTER PLATES** 25 26 27



**1. Angle Adjustment**

- a. Loosen the eccentric bolts (B) that secure the caster assembly.
- b. Place a large right triangle against the flat surface and the front surface of the caster bearing housing. This will align the caster stem rotational axis perpendicular to the flat surface.
- c. Align and retighten the two eccentric bolts.
- d. Repeat the process for the other caster.

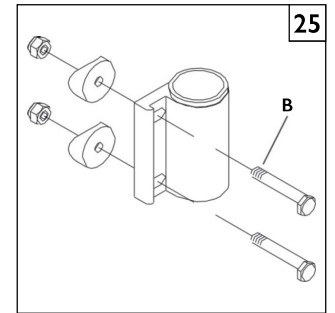

**2. Angle Adjustment: Caster Plates with Transit Option** 27

- a. Loosen the bolts (C) that secure the caster assembly.
- b. Place a large right triangle against the flat surface and the front surface of the caster bearing housing. This will align the caster stem rotational axis perpendicular to the flat surface.
- c. Align the eccentric washers and retighten the bolts.
- d. Repeat the process for the other caster.

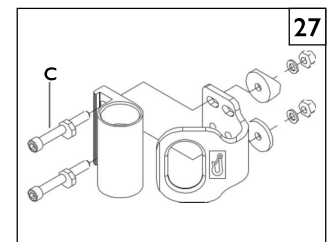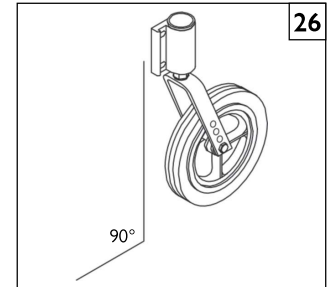


**U. ANTI-TIP TUBES - REAR AND FRONT** 28 29

**NOTE** – Sunrise recommends anti-tip tubes for all wheelchairs.


**1. To Insert Anti-Tip Tubes into Receivers** 28

- a. Press the anti-tip release button (D) on the anti-tip tube so that the release pin (E) is drawn inside.
- b. Insert the tube into the anti-tip receiver.
- c. Turn the anti-tip tube until release pin is positively engaged in the receiver mounting hole (F).
- d. Insert second anti-tip tube the same way.

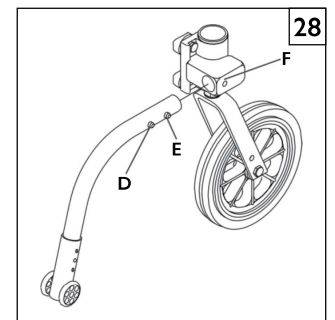

**2. To Adjust Anti-Tip Height** 29

- a. Press the anti-tip wheel release button (G) so the release pin is drawn inside.
- b. Raise or lower to one of the three pre-drilled holes.
- c. Turn the anti-tip tube until release pin is positively engaged in the receiving hole (H).
- d. Adjust the second anti-tip tube wheel the same way. Both wheels should be exactly the same height.

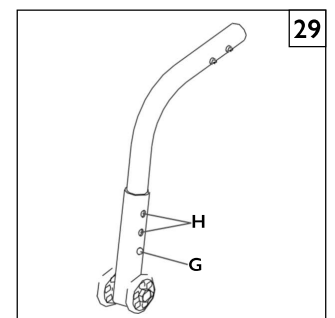



## V. REAR WHEEL ADJUSTMENTS

### 1. Wheelbase Adjustment 30 31

- a. Remove rear wheel.
- b. Remove the four inside nuts (A) located at the top and bottom of the universal plate (B).
- c. Lift the axle plate off and re-position in desired location.
- d. Replace and tighten nuts and replace wheel.
- e. Repeat on other side of chair.


**NOTE** – Adjusting the wheelbase affects the center of gravity. Moving the axle forward will increase your turning speed and lighten the front end. The further back the axle, the more stable the chair becomes. In addition, the axle plate may be placed in reverse position to gain additional forward or back positioning.

Use a torque setting of 100 in.-lbs. when making this adjustment.

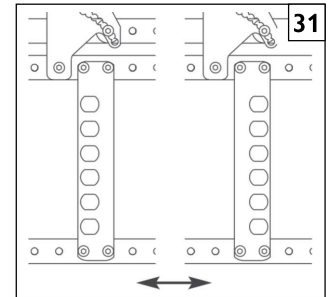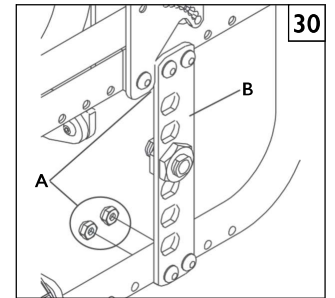

### 2. Wheel Camber Adjustment 32


- a. Loosen top fasteners (C) attaching the axle plate to the frame.
- b. Remove the bottom fasteners (D).
- c. Add or subtract camber spacers (E) for more or less camber.
- d. Replace and tighten bottom fasteners. Tighten top fasteners.

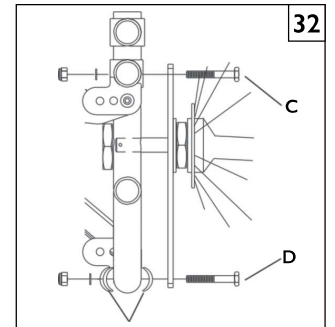

### 3. Wheelbase Width Adjustment 33

- a. Loosen the nut attached to the axle plate (F).
- b. Slide the axle sleeve through the axle plate.
- c. Tighten the nut at the other end of the axle sleeve.
- d. Repeat on other side of chair. The axle should protrude equal amounts on both sides of the chair.



**NOTE** – Adjusting the wheelbase width allows the rider the option to move the wheels closer or further away from the hips. It also compensates for camber adjustment and gives the proper wheel spacing to maximize pushing efficiency. To achieve the most narrow chair possible, axle plates can be mounted on inside frame tubes. It may be necessary to re-adjust the axle sleeves. Use a torque setting of 100 in.-lbs. when making this adjustment.

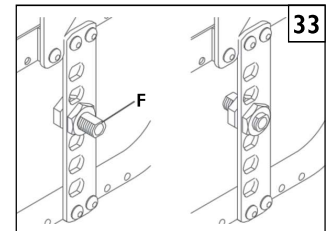

### 4. Seat Height Adjustment 34 35

- a. Remove the axle nut (H) on the inside of the axle plate.
- b. Slide the axle assembly (G) with bolts vertically on the axle plate to the desired height. (Total range 6.5").
- c. Replace and tighten axle nuts.

**NOTE** – Seat height adjustments allow the rider to adjust the seat for environment, body and mechanical efficiency. A front caster adjustment should be made to correspond with any change in seat height. Use a torque setting of 100 in.-lbs. when adjusting rear axle plate fasteners. In addition, the axle plate may be rotated vertically to gain additional seat height adjustment.

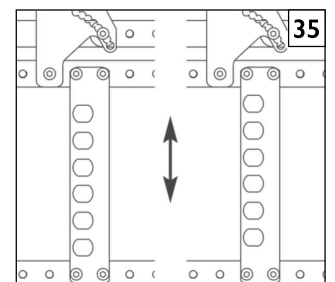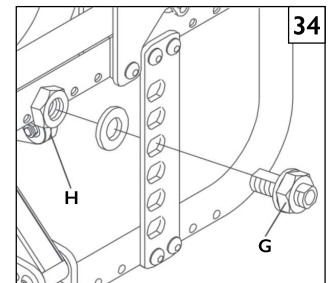

**W. GROWING SEAT PAN 36 37**

**1. Removing and Attaching the Seat Pan**

- a. To remove, grasp front of seat pan where indicated in Figure 36 and firmly pull upward until seat mounting clips disengage from the frame.
- b. To reattach, position seat pan on frame rail as desired and push down firmly on seat pan where it is attached to the seat mounting clips. Be sure to engage all four clips on frame rails.

**NOTE** – Rear of seat pan will be labeled.



**WARNING**

**Check for engagement by pulling up lightly on seat pan. It should not move upward.**

**2. Adjusting Seat Pan Width**

- a. Remove the two screws that secure each seat mounting clip to the seat pan 36.
- b. Position the seat mounting clips to the desired position 37, reattach the screws. Each hole is .5" apart for a total of 2" of width adjustment range.

**NOTE** – Tighten screws to 100 in. lbs. torque.



- c. Repeat steps 1 and 2 for each mounting clip.

**NOTE** – Both sides must be mounted to same hole position.

**3. Adjusting Seat Pan Depth**

- a. Loosen the two lock nuts on under side of seat pan that attaches front seat pan panel to rear seat pan panel 36.
- b. Reposition seat pan to the desired length 37, and reattach the lock nuts.

**NOTE** – Tighten lock nuts to 100 in. lbs. torque.

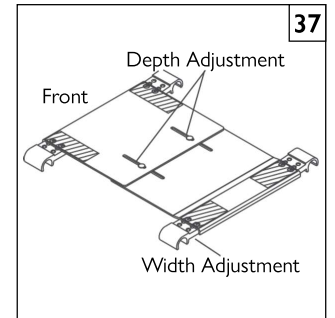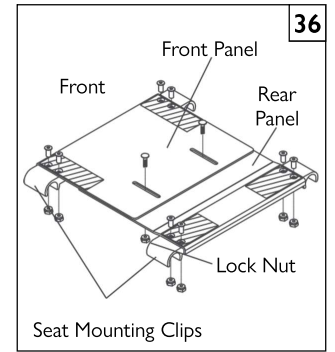

**X. ADJUSTABLE BACK UPHOLSTERY**

**1. Adjusting the Back Upholstery Width 38**

- a. With the upholstery on the back canes, lift the back upholstery flap from the rear 38 to gain access to the Velcro® closure straps.
- b. Release each of the individual adjustment straps by opening the Velcro closure straps 38.
- c. Lengthen or shorten the straps as needed to achieve desired width.
- d. Reattach the Velcro closure straps on each strap. Be sure to firmly engage Velcro® over the entire available strap length.

**NOTE** – Contours to match a client's needs may be developed by adjusting the Velcro closure straps to varying lengths.



**WARNING**

**Velcro® must overlap by at least 2".**



**2. Adjusting Backrest Height 39**

- a. Remove button head cap screw that attaches the push handle cane to the adjustment tube 39.
- b. Reposition push handle cane in adjustment tube to desired height and realign holes.

**NOTE** – Each hole raises or lowers backrest height by 1". A total of 4" height adjustment possible.

- c. Re-insert button head cap screw.
- d. Tighten button head cap screw. Check to be sure that push handle cane is securely attached to adjustment tube.


**NOTE** – Tighten button head cap screw to 60 in. lbs. torque.

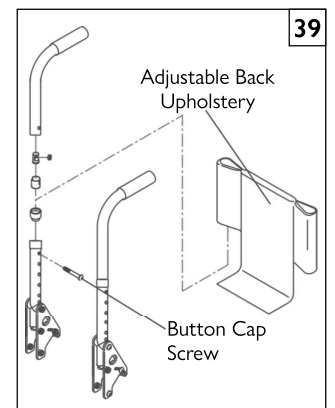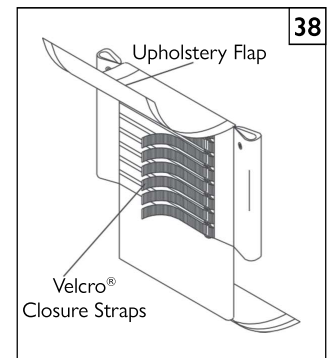

**Y. SOLID BACK AND SEAT INSERT** 40 41


**1. Attaching/Removal** 40

- a. Position seat and back on the frame.
- b. Position clamps on the seat base and backposts.
- c. Insert the quick-release pins (A) through the mounting hole on the hook clamps.
- d. Remove the seat by reversing the above process.

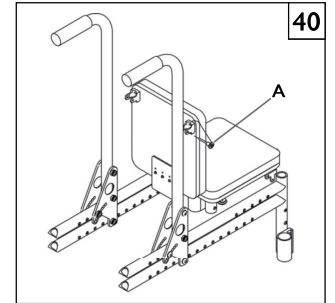

**2. Attaching/Removal of Transit Option Back/Seat** 41

- a. Position seat and back on the frame.
- b. Position clamps on the seat base and backposts.
- c. Insert the quick-release pins (B) through the mounting holes on both the base and backposts.
- d. Remove the seat by reversing the above process.

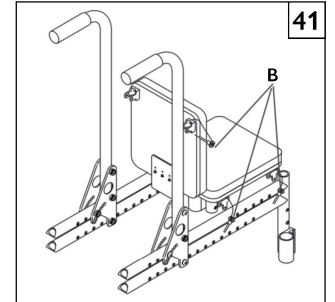


**Z. SEAT DEPTH** 42

**1. Adjustment**

- a. Remove the bolts securing the backrest pivot plate (C) onto the seat frame.
- b. Position the backrest pivot plate in the pre-drilled holes on the frame.
- c. Replace bolts and tighten securely.

**NOTE** – This adjustment may also require repositioning the seat-to-back bracket (under the solid seat) into the second set of mounting holes.

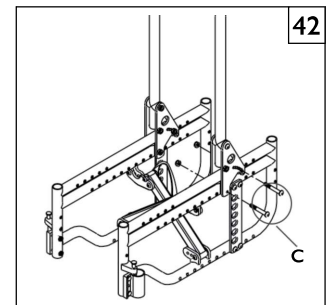

**AA. SEAT TILT** 42

**1. Adjustment**


- a. Remove the rear bolts (C) from the backrest pivot plate.
- b. Tilt the back posts to the desired tilt position.
- c. Replace and tighten bolts.

**BB. CROSS BRACE ASSEMBLY** 43

**1. Adjustment**

- a. Remove the four bolts (D) that attach the cross brace to the side frames.
- b. Move the cross brace to the desired location on the tabs (E).
- c. Replace and tighten the bolts.

**NOTE** – Align the bolts through holes in the tabs that correspond on each side. Re-tighten bolts to 100 in.-lbs. Cross braces have 2" of width growth.

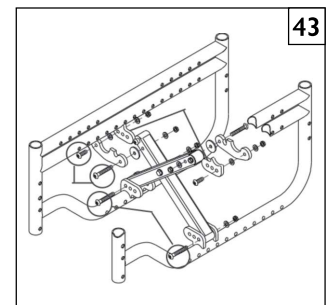


**CC. STRUT TUBE ASSEMBLY** 44

**NOTE** – Strut Tubes are installed on rigid chairs.

**1. Width Adjustment**

- a. Remove the saddle bolts (F) from each end of the strut tubes (G) for all three strut tubes.
- b. Slide the adjustable extension to the desired width.
- c. Replace and tighten the saddle bolts in the strut tubes.

**NOTE** – Strut tubes have 2" width adjustment available. Size changes may require a larger seat insert and footplates. Call your authorized Sunrise supplier for details. Use a torque setting of 100 in.-lbs. when making this adjustment.

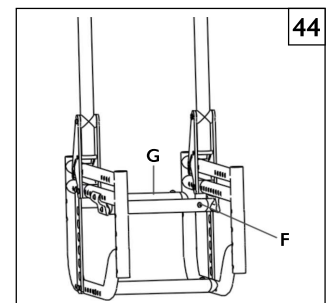


**DD. FOLDING THE CHAIR** 45

**NOTE** – Only chairs with cross brace assembly will fold.

**1. Folding/Opening**

- a. To fold the chair, remove the seat and back.
- b. Pull the cross brace strap (A) up.
- c. Press the side frames together firmly.
- d. To open the chair, extend fingers and apply even pressure with palm of hand directly on decals or cross brace until it is fully extended.
- e. Replace the seat and back.
- f. To avoid pinching, be careful when folding or unfolding a chair.

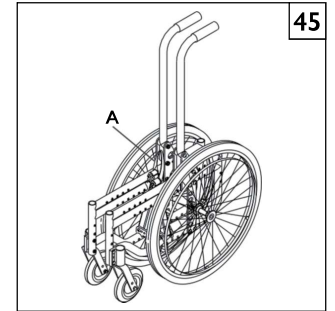**EE. CHECK-OUT**

After the wheelchair is assembled and adjusted, it should roll smoothly and easily. All accessories should also perform smoothly. If you have any problems, follow these procedures:

1. Review assembly and adjustment sections to make sure chair was properly prepared.
2. Review troubleshooting guide.
3. If your problem persists, contact your authorized Sunrise supplier. If you still have a problem after contacting your authorized Sunrise supplier, contact Sunrise customer service. See the introduction page for details on how to contact your authorized Sunrise supplier or Sunrise customer service.

You will need to adjust your chair from time to time for best performance (especially if you alter the original settings). This chart gives you a first solution, then a second and a third if needed. You may need to look farther to find the best solution for a specific problem.

**NOTE** – To keep track of your progress, make only *ONE* change at a time.

SYMPTOMS						SOLUTIONS
Left turn in chair	Right turn in chair	Looseness in chair	Sluggish turning	Squeaks and rattles	Caster flutter	
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>			Make sure tire pressure is correct and equal in both rear tires and front caster tires, if pneumatic.
		<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	Make sure all nuts and bolts are snug.
		<input checked="" type="checkbox"/>		<input checked="" type="checkbox"/>		Make sure all spokes and nipples are tight on radial spoke wheels.
				<input checked="" type="checkbox"/>		Use Tri-Flow Lubricant (Teflon™-based) between all modular frame connections and parts.
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		<input checked="" type="checkbox"/>		<input checked="" type="checkbox"/>	Check for proper caster plate adjustment. See instructions for caster plate adjustment.
	<input checked="" type="checkbox"/>					Add a spacer washer between the plastic saddle and the caster plate on the top of the left caster plate.
<input checked="" type="checkbox"/>						Add a spacer washer between the plastic saddle and the caster plate on the top of the right caster plate.
<input checked="" type="checkbox"/>						Add a spacer washer between the plastic saddle and the caster plate on the bottom right caster plate retaining bolts.
	<input checked="" type="checkbox"/>					Add a spacer washer between the plastic saddle and the caster plate on the bottom left caster plate retaining bolts.
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>		<input checked="" type="checkbox"/>			Make sure there is an equal number of camber washers on each axle plate.
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>				<input checked="" type="checkbox"/>	Make sure both front casters touch the ground simultaneously. If they do not, add the proper spacers between the bottom bearing of the caster plate and fork stem nut until they do. Make sure you check for this problem on a flat surface.

**A. INTRODUCTION**

1. Proper maintenance will improve performance and extend the useful life of your chair.
2. Clean your chair regularly. This will help you find loose or worn parts and make your chair easier to use.
3. Repair or replace loose, worn, bent or damaged parts **before using this chair**.
4. To protect your investment, have all major maintenance and repair work done by your authorized supplier.

**B. MAINTENANCE CHART**

Weekly	3 Months	6 Months	Annually	CHECK...
<input checked="" type="checkbox"/>				Tire inflation level
<input checked="" type="checkbox"/>				Wheel locks
	<input checked="" type="checkbox"/>			All fasteners for wear and tightness
		<input checked="" type="checkbox"/>		Armrests
		<input checked="" type="checkbox"/>		Axle and axle sleeves
	<input checked="" type="checkbox"/>			Quick-release axles
	<input checked="" type="checkbox"/>			Wheels, tires and spokes
	<input checked="" type="checkbox"/>			Casters
	<input checked="" type="checkbox"/>			Anti-tip tubes
		<input checked="" type="checkbox"/>		Frame
	<input checked="" type="checkbox"/>			Upholstery
			<input checked="" type="checkbox"/>	<b>Service by authorized supplier</b>

**C. MAINTENANCE TIPS**

**1. Axles & Axle Sleeves:**

Check axles and axle sleeves every six months to make sure they are tight. Loose sleeves will damage the axle plate and will affect performance.

**2. Tire Air Pressure:**

Check air pressure in pneumatic tires at least ONCE A WEEK. The wheel locks will not grip properly if you fail to maintain the air pressure shown on tire sidewall.

**D. CLEANING**

**1. Paint Finish**

- a. Clean the paint finish with mild soap at least once a month.
- b. Protect the paint with a coat of non-abrasive auto wax every three months.

**2. Axles and Moving Parts**

- a. Clean around axles and moving parts WEEKLY with a slightly damp (not wet) cloth.
- b. Wipe off or blow away any fluff, dust, or dirt on axles or moving parts.

*NOTE – You do not need to grease or oil the chair.*

**3. Upholstery**

- a. Hand-wash (machine washing may damage fabric).
- b. Drip-dry only. DO NOT machine dry as heat will damage fabric.

**E. STORAGE TIPS**

1. Store your chair in a clean, dry area. If you fail to do so, parts may rust or corrode.
2. Before using your chair, make sure it is in proper working order. Inspect and service all items on the “Maintenance Chart.”
3. If stored for more than three months, have your chair inspected by an authorized supplier before use.



**A. FOR FOR LIFETIME**

Sunrise warrants the frame and cross brace if applicable of this wheelchair against defects in materials and workmanship for the life of the original purchaser.

The expected life of this frame is five years.

**B. FOR ONE (1) YEAR**

We warrant all Sunrise-made parts and components of this wheel-chair against defects in materials and workmanship for one year from the date of first consumer purchase.

**C. LIMITATIONS**

1. We do not warrant:
  - a. Tires and tubes, upholstery, pads and push handle grips.
  - b. Damage from neglect, accident, misuse, or from improper installation or repair.
  - c. Products modified without Sunrise Medical's express written consent.
  - d. Damage from exceeding the weight limit.
2. This warranty is VOID if the original chair serial number tag is removed or altered.
3. This warranty applies in the USA only. Check with your supplier to find out if international warranties apply.
4. This warranty is not transferable and only applies to the first consumer purchase of this wheelchair through an authorized Sunrise Medical dealer.

**D. WHAT WE WILL DO**

Our sole liability is to repair or replace covered parts. This is the exclusive remedy for consequential damages.

**E. WHAT YOU MUST DO**

1. Obtain from us, while this warranty is in effect, prior approval for return or repair of covered parts.
2. Return the wheelchair or part(s), freight pre-paid to the designated Sunrise Medical location.
3. Pay the cost of labor to repair, remove or install parts.

**F. NOTICE TO CONSUMER**

1. If allowed by law, this warranty is in place of any other warranty (written or oral, express or implied, including a warranty of merchantability, or fitness for a particular purpose).
2. This warranty gives you certain legal rights. You may also have other rights that vary from<sup>1</sup> state to state.

Record your serial number here for future reference:

---



A series of horizontal lines for writing, starting from the first line below the pen nib and extending down to the bottom of the page.



A series of horizontal lines for writing notes, spanning the width of the page. The lines are evenly spaced and extend from the left margin to the right margin, starting from the first line below the pen nib and continuing down to the bottom of the page.

Sunrise Medical S.r.l.  
Via Riva, 20 – Montale  
29122 Piacenza  
Italia  
Tel.: +39 0523 573111  
www.SunriseMedical.it

Sunrise Medical AG  
Erlenauweg 17  
CH-3110 Münsingen  
Schweiz/Suisse/Svizzera  
Tel +41 (0)31 958 3838  
www.SunriseMedical.ch

Sunrise Medical AS  
Delitoppen 3  
1540 Vestby  
Norge  
Telefon: +47 66 96 38 00  
post@sunrisemedical.no  
www.SunriseMedical.no

Sunrise Medical AB  
Neogatan 5  
431 53 Mölndal  
Sweden  
Tel.: +46 (0)31 748 37 00  
post@sunrisemedical.se  
www.SunriseMedical.se

MEDICCO s.r.o.  
H – Park, Heršpická 1013/11d,  
639 00 Brno  
Czech Republic  
Tel.: (+420) 547 250 955  
Fax: (+420) 547 250 956  
www.medicco.cz  
info@medicco.cz  
Bezplatná linka 800 900 809

Sunrise Medical Aps  
Mårkærvej 5-9  
2630 Taastrup  
Denmark  
+45 70 22 43 49  
info@sunrisemedical.dk  
www.SunriseMedical.dk

Sunrise Medical Australia  
11 Daniel Street  
Wetherill Park NSW 2164  
Australia  
Ph: +61 2 9678 6600  
Email: enquiries@sunrisemedical.com.au  
www.SunriseMedical.com.au

Sunrise Medical (US) LLC  
North American Headquarters  
12002 Volunteer Blvd.  
Mount Juliet, TN 37122, USA  
(800) 333-4000  
(800) 300-7502  
www.SunriseMedical.com



© 2025 Sunrise Medical (US) LLC  
930377 Rev. G

Sunrise Medical GmbH  
Kahlbachring 2-4  
D-69254 Malsch  
Deutschland  
Tel.: +49 (0) 7253/980-0  
Fax: +49 (0) 7253/980-222  
www.SunriseMedical.de

Sunrise Medical Ltd.  
Thorns Road  
Brierley Hill  
West Midlands  
DY5 2LD  
England  
Phone: 0845 605 66 88  
Fax: 0845 605 66 89  
www.SunriseMedical.co.uk

Sunrise Medical S.L.  
Polígono Bakiola, 41  
48498 Arrankudiaga – Vizcaya  
España  
Tel.: +34 (0) 902142434  
Fax: +34 (0) 946481575  
www.SunriseMedical.es

Sunrise Medical Poland  
Sp. z o.o.  
ul. Elektronowa 6,  
94-103 Łódź  
Polska  
Telefon: + 48 42 275 83 38  
Fax: + 48 42 209 35 23  
E-mail: pl@sunrisemedical.de  
www.Sunrise-Medical.pl

Sunrise Medical B.V.  
Defensiedok 20  
3433 NE Nieuwegein  
The Netherlands  
T: +31 (0)30 – 60 82 100  
F: +31 (0)30 – 60 55 880  
E: info@sunrisemedical.nl  
www.SunriseMedical.nl

Sunrise Medical HCM B.V.  
Vossenbeemd 104  
5705 CL Helmond  
The Netherlands  
T: +31 (0)492 593 888  
E: customerservice@sunrisemedical.nl  
www.SunriseMedical.nl  
www.SunriseMedical.eu (International)

Sunrise Medical S.A.S  
ZAC de la Vrillonnerie  
17 Rue Mickaël Faraday  
37170 Chambray-Lès-Tours  
Tel : + 33 (0) 2 47 55 44 00  
Email: info@sunrisemedical.fr  
www.SunriseMedical.fr

Sunrise Medical Canada Inc.  
1000 Creditstone Rd., Unit #2  
Concord, ON, L4K 4P8  
Canada  
Phone: 1-800-263-3390  
Fax: 1-800-561-5834  
www.SunriseMedical.ca