# JAY<sup>®</sup>

# ZIP BACK

## JAY<sup>®</sup> Zip Back

### IMPORTANT CONSUMER INFORMATION

**NOTICE:** This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

**SUPPLIER:** This manual must be given to the user of this product.

**USER:** Before using this product, read this entire manual and save for future reference.

**WARRANTY REGISTRATION:** To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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Manuel d'instructions

**ENGLISH**

Introduction .....	3
Maintenance and Cleaning .....	4
Fitting the Back to the Chair.....	5
Installation .....	6
Adjustment.....	7
Transit Safety .....	8
Removal and Replacement.....	9
Warranty .....	9

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Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a Zip Back is appropriate. Backs should only be installed by an authorized Sunrise Medical supplier.

## THE JAY ZIP BACK

The JAY Zip Back is designed to provide the important benefits of a proper back support for a wide variety of pediatric users. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to use.

**Maximum user weight: 165 lbs**

### **⚠ WARNING**

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.

Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.


### **⚠ WARNING**

Do not use the back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

### **⚠ WARNING**

Do not use the back handle to lift wheelchair. It is intended only for back removal.

## ZIP BACK

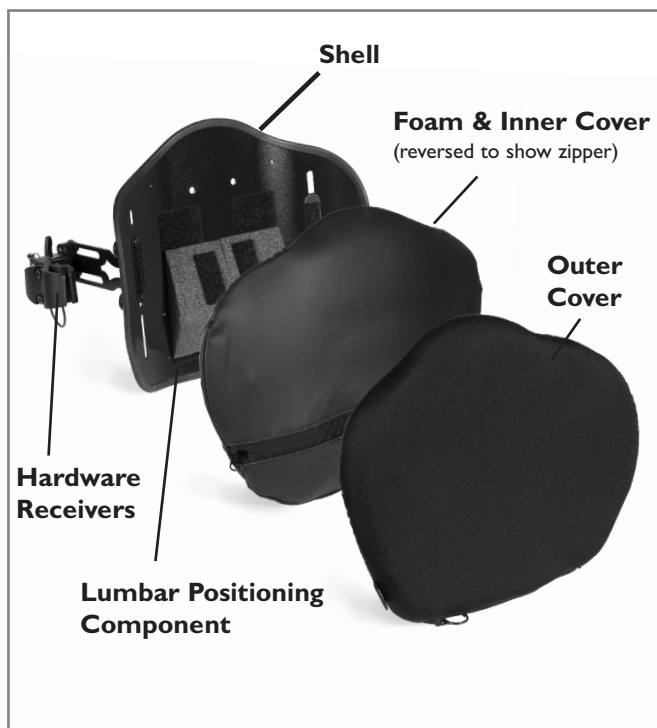**MAINTENANCE**

Sunrise recommends that all fasteners be checked for wear, such as loose bolts or broken components every 6 months. Loose fasteners should be retightened according to the installation instructions. All fasteners should be tightened to torque specifications as stated in the hardware installation section. Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose or broken components.

**NOTE-** Wrench has been included with the back for your convenience.

**EASY CLEANING**

**NOTE-** When disassembling the back for cleaning, note the position of the Velcro® Lumbar Positioning Components for correct reassembly.

**To clean back cover**

Remove cover by detaching Velcro. Detach cover from back shell. Locate zipper on bottom of the cover and unzip. Remove foam with inner cover and turn cover inside out. Zip cover closed to avoid snagging. Machine wash in warm water (50°C). Drip or tumble dry on low heat.

**NOTE:** Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

Reinsert the foam with inner cover into the cover. Please note, the base has two layers of foam. When reinserting the foam, ensure that the thinner, firmer, light grey foam is facing the shell and the softer foam is facing the user. Once the foam is secured inside the cover, zip cover closed.

**To clean back shell and Inner Cover**

Shell and inner cover may be wiped clean with soap and water. No other maintenance should be required.

**To clean positioning components**

Positioning components can be wiped clean with soap and water. Do not immerse positioning components underwater.

If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

Laundry Care Symbols	
	Machine wash in warm water (50° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

## DETERMINING COMPATIBILITY

### Wheelchair Types

The Zip Back is designed to be compatible with most wheelchairs with the following exceptions:

Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.

### **⚠ WARNING**


**Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.**

### Wheelchair Dimensions

The JAY Zip back is designed to replace the wheelchair's sling upholstery. Prior to installation determine if the wheelchair used has compatible back canes, by measuring the width between your back canes as seen in Figure 1. Then using that measurement, check Table 1 to determine if the back will fit your chair.

Next check that the back canes are a compatible diameter. The JAY back hardware can be mounted on wheelchair backcanes ranging from 3/4" to 1 1/8". Measure your back canediameter (Fig. 2) and refer to Table 2 to ensure your backcanes are compatible.


If you cannot determine that the back canes are compatible please contact your authorized Sunrise Medical provider or Sunrise Medical customer service.

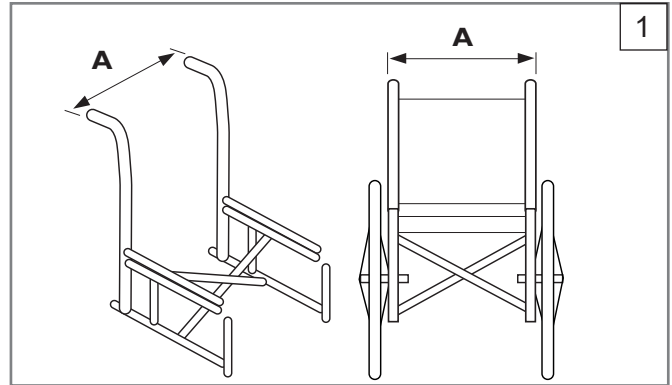**Table 1**

Zip Back Width* (all contours)	Width of back canes from outside to outside of the tubes (Fig. 1-A)			
	Minimum width		Maximum width	
	inch	cm	inch	cm
8"	7.7	19.6	9.3	23.6
10"	9.7	24.6	11.3	28.7
12"	11.7	29.7	14.3	36.3
14"	13.7	34.9	16.3	41.3

\* Back Width is identified in your back's part number.  
Example: JZIPB10S is a 10" wide back.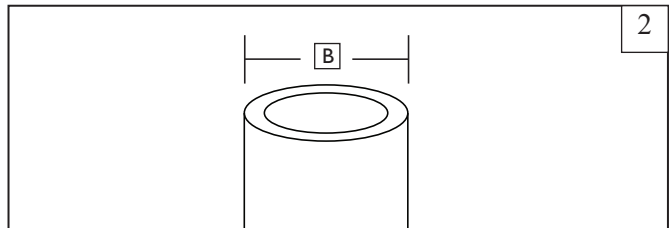

Back Width (all contours)	Cane diameter (B)			
	Minimum width		Maximum width	
	inch	cm	inch	cm
8" - 14"	3/4	1.9	1 1/8	2.85

## A. JAY (RH) HARDWARE

Please read the following instructions before beginning the installation. To install the Zip back, with RH hardware, it is best to begin without a user in the wheelchair. Once the back has been installed the user can be seated back in the wheelchair for minor adjustments.

### 1. Tools required (included with the JAY Zip back):

- 4mm hex key
- 10mm box end wrench

## B. BACK HARDWARE INSTALLATION 3 4 5 6

### 1. Remove existing back components

- Remove the existing wheelchair back according to the wheelchair manufacturer's instructions.
- Remove any pre-existing back hardware or components.

### 2. Locating the hardware receivers

- The backshell should come with mounting brackets (A) attached.
- Visually determine the optimum clamp receiver location (B) on the back canes. The mounting hardware should be at approximately equal heights on each back post (Fig. 3), and parallel to the seat frame (Fig. 4).

**NOTE**—A higher location on the back canes are generally better.

### 3. Attaching the Hardware receivers

- Using the 4mm hex key, Loosen the clamp screws (Fig. 5) (C)





**NOTE**—For Large tube diameters, the clamp screws may have to be removed completely.

- Attach and align the receiver to the wheelchair back canes (D).
- Hand tighten the clamp screws (C).
- The receivers may require additional lateral adjustments to ensure proper alignment.

### 4. Attaching and aligning the Zip Back shell.

- Using a 10mm wrench, loosen the bracket nuts (E), and mounting pin bolts (H) until the hardware can move easily in all directions.
- Attach the Zip Back to the hardware receivers by inserting left and right mounting pins (F) in the left and right receivers (G).
- Locate the back to the approximate location, relative to both the chair and the user.
- Tighten bolts on the inside of the mounting pin (F). The recommended torque specification is 100-110 in/lbs. (11.3-12.4 Nm)
- Once all components have been properly aligned, tighten the clamp screws (C) to approximately 75-85 in-lbs (8.5-9.3 Nm).
- Finish by inserting the locking pin into the receiver, and releasing a few times to ensure smooth operation.

**NOTE**—Back depth and angle may require additional adjustment to ensure proper fit to the user.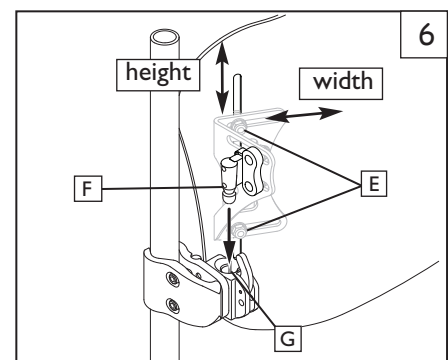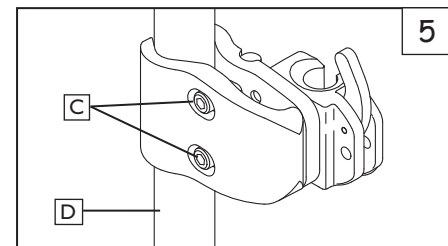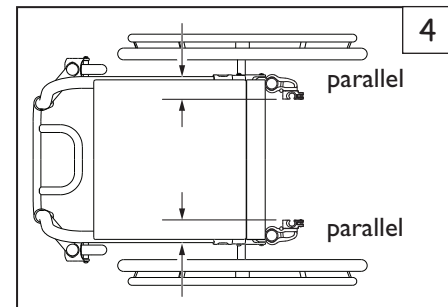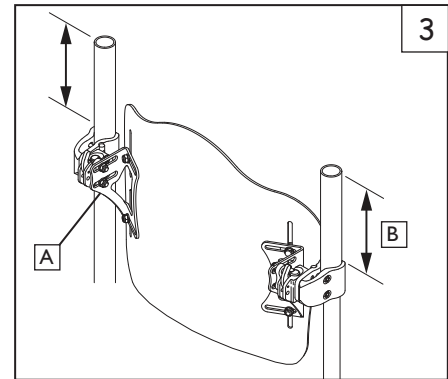

### C. ADJUSTING HEIGHT, DEPTH, AND ANGLE 7

Once the initial installation is complete, adjustments can be made with the user seated in the chair for a personalized fit. Make sure that the user does not apply full to the back until the hardware has been fully tightened to the specified torque requirements.


#### 1. Height adjustment

- a. With the user seated in the chair, identify the correct height for the back.
- b. Have the user slide forward slightly, and using a 10mm wrench loosen the nuts ( I ).
- c. Move back to the correct height, and tighten the nuts to 75-85 in-lbs, or (8.5-9.3 Nm)

#### 2. Depth and angle adjustment

Once the initial installation is complete, adjustments can be made with the user seated in the chair for a personalized fit. Hardware allows for 2 inches of anterior depth adjustment, 2.75 inches with reversing right and left mounting pins.

- a. Using a 10mm wrench, loosen the left and right hex screws ( J ) on the bracket slots.
- b. Adjust the depth and/or angle settings and securely tighten the screws to lock in the adjustment. Recommended torque specification is 100-110 in-lbs or (11.3 -12.4 Nm).
- c. If additional depth is needed, remove the J3 back from the receivers. Remove the screws(K) from the receiver using a 4mm hex key and reposition the hardware to the desired depth.
- d. After adjusting to the correct depth, tighten the screws to 25-35 in-lbs (2.8 - 4 Nm).

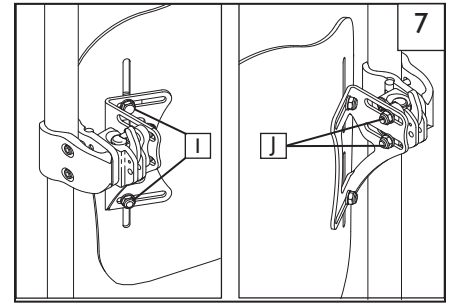### E. CHECK OUT

#### WARNING

Ensure that all hardware has been properly tightened according to suggested specifications.

- a. Make sure the Back Support is locked, and the locking pin is inserted into the receiver.
- b. Check that the Back Support can be attached smoothly and securely, by attaching and removing the assembly several times.
- c. Double check fit and comfort for the user before putting the Back Support into use.
- d. Failure to follow this check out procedure could result in injury and/or damage to your Back Support and/or components.

## BACK TRANSIT SAFETY

This back has been dynamically tested for use in a motor vehicle. Please follow all installation, use and maintenance instructions within this manual as well as the transit instructions listed below.

### WARNINGS

1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraints.
2. The distance between the top of the user's shoulder and the top of the back shell should not exceed 3" (7.6 cm).
3. The wheelchair must be labeled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
4. If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
5. The Wheelchair Tiedown and Occupant Restraint System (WTORS) should be compatible with the specific wheelchair, used as indicated in the manufacturer's instructions and should comply with the performance requirements of ISO Standard 10542.
6. The wheelchair must be forward facing during transport.
7. In order to reduce the potential of injury to vehicle occupants wheelchair-mounted accessories such as trays and respiratory equipment should be removed and secured separately.
8. This product is intended for use by users ranging in weight from 25 to 165 lbs.
9. Postural support devices such as pelvic positioning belts, anterior trunk supports or postural supports such as lateral trunk supports should not be relied on for occupant restraint in a moving vehicle unless they are labeled as conforming to ISO Standard 16840-4.
10. Alterations or substitutions should not be made to the structure, parts or components, without consulting the manufacturer.

### WARNING

**FAILURE TO HEED THESE WARNINGS  
COULD CAUSE SEVERE INJURY TO THE  
WHEELCHAIR RIDER OR OTHERS.**

## REMOVAL AND REPLACEMENT

### Removal

Following installation and adjustment, the back can be easily removed from the wheelchair if necessary. To remove, pull down on back handle, rotate up, then remove from the hardware receivers.

### Proper reattachment after removal

To reattach the back to the hardware receivers, pull down on cord, insert back into receivers, and rotate down.

### **WARNING**

**If you are uncertain if you have heard the locking “click”, pull up on the back to ensure that it has been securely locked.**

## WARRANTY

### 24 MONTH LIMITED WARRANTY

Each JAY Zip Back is carefully inspected and tested to provide peak performance. Every JAY Zip Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back's removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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© 2025 Sunrise Medical (US) LLC
Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S
Sunrise Medical Canada Inc.